

# Colectomy


## Trocars and ViKY positioning

### Top view

For optimal surgeon ergonomics, position the passive arm on the opposite side of the surgeon, towards the shoulders, with the passive arm extending distally to the abdomen as illustrated.

- Optical trocar
- Operating trocar

Note: position of trocars are given as a rough guide.

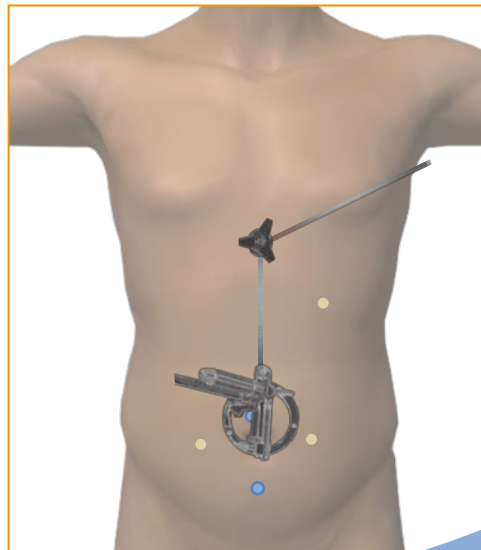

### Side view



*Optimal ring positioning for 0° laparoscope*

To obtain same field of view with a 30° laparoscope, the ring is angulated of an additional 30°. The position and angulation of the ViKY base can be quickly and easily adjusted as needed during the procedure to maintain optimal visualization of the targeted surgical site.



*Optimal ring positioning for 30° laparoscope*



For an optimal use of ViKY, camera must be aligned with "up"/"down" plane as shown on picture just beside.

[www.endocontrol-medical.com](http://www.endocontrol-medical.com)

EndoControl S.A.S. – 5 avenue du Grand Sablon - 38700 La Tronche – France  
Tel : +33 (0)4 76 63 75 83 / Fax : +33 (0)4 76 54 95 61