


Inguinal Hernia

(Left)

Trocars and ViKY positioning

Top view

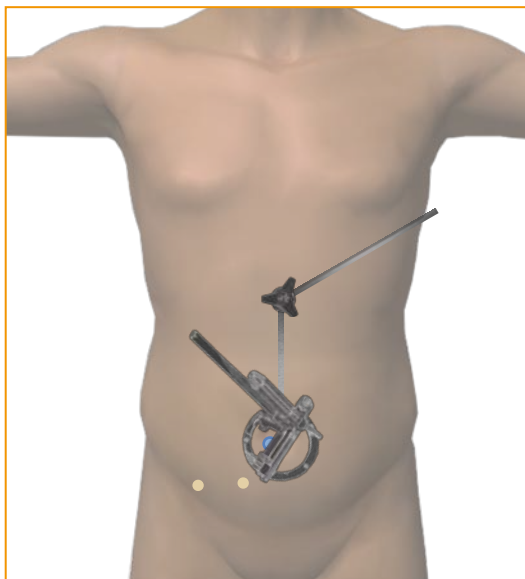

For optimal surgeon ergonomics, position the passive arm on the opposite side of the surgeon, towards the shoulders, with the passive arm extending distally to the abdomen as illustrated.

- Optical trocar
- Operating trocar

Note: position of trocars are given as a rough guide for left hernia.

Side view

For optimal visualization of the surgical site, angle the base of the ViKY platform as illustrated with the caudal section of the ring slightly elevated. The position and angulation of the ViKY base can be quickly adjusted as needed during the procedure to maintain optimal visualization of the targeted surgical site.

